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IMPLANTES DENTOFLEX

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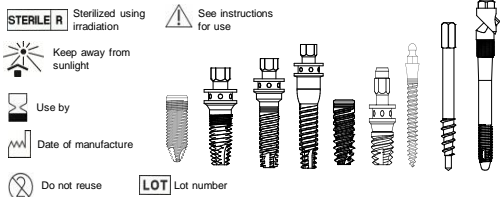
Reg. ANVISA Nº 80100330011

Instructions for Use:
DENTOFLEX IMPLANTS
Bone-integrable Dental Implant

BONE compressive - suitable for deployments made immediately after tooth extraction and immediate function.
Design Platform: IH, CM and TISW.
Series: Pre-Mounted Series and Series Regular
BONE EXPANSIVE - Suitable for deployments made immediately after tooth extraction and immediate function, is easy to thread, because it presents anatomical shape closer to the natural tooth root.
Design Platform: CM.
Series: Series Regular

SERIES: Gold Series: Its use is recommended to make faster surgery and need, with golden assembler that facilitates viewing the level of the bone crest.

Pre-assembled series: Its use is recommended to make faster and more accurate surgery.

Regular series: Its use is indicated to optimize the placement procedure, the handling of the implant during installation in surgery should be only with titanium instruments or coated with titanium nitride.

COMPOSITION:

Gold Series: implant, cover screw, golden assembler and bolt assembler.
Pre-Mounted series: implant, cover screw, fitter and bolt assembler.
Regular series: implant and cover screw.

INSTRUMENTAL:

Cylindrical, conical, Osseo Compressive, Flex and Osseo Expansive: You must use the Surgical Kit for Dental Implant Mix DENTOFLEX Bicortical: pipe wrench 45 degrees; Socket wrench 90 degrees; Key to wrench; Folders; and Torque Wrench Surgical. Temporary: You must use Helical Drill 1.8mm, Digital Key to 2.40 mm for O-ring for prosthesis or Key Torque Wrench with the aid of a torque wrench and Ratchet or T. Zygomatic Key: You must use the Surgical Kit zygomatic implant DENTOFLEX. Short: You must use surgical kit for Implant Short DENTOFLEX. All instruments are sold separately. The torque (N / cm) suitable for model Implants: Bicortical must be at least 45N, Temporary must be at least 35N, Zygomatic must be at least 45N, cylindrical, tapered and Bone Compressive below is the minimum torque for each diameter implant (mm).

(EH, IH, CM, FL, OC and XP)
Ø2,95-30N / Ø3,25-32N / Ø3,25M-32N / Ø3,50-35N / Ø3,75-40N / Ø4,00-45N / Ø4,25-45N / Ø4,30-45N / Ø4,50-50N / Ø5,00-60N / Ø6,00-60N.

DECK DESIGN NOTE:

Internal Hex (IH, HISW): Suitable for single implants and small fixed prosthesis bolted or cemented 2-5 teeth, removable dentures with at least 2 implants or fixed prostheses cemented up to 5 teeth. **External Hexagon (EH, HESW) and Internal Torque (IT TISW):** Suitable for single implants, dentures with at least 2 implants, cemented or screwed fixed prosthesis 2-5 teeth and screwed extensive fixed prosthesis over 5 teeth. **Cone Morse (CM):** Suitable for single implants and small fixed prosthesis bolted or cemented 2-5 teeth, removable dentures with at least two teeth, cemented or screwed fixed prosthesis up to 2-5 teeth and screwed extensive fixed prosthesis over 5 teeth. **Flex (FL):** Suitable for unitary implants and screwed fixed prostheses, dentures (over-denture) at least two implants and absence of micro spaces.

Surgery of DENTOFLEX implants should be performed under control conditions and surgical sterilization, should aim for a strong and direct connection between the bone and the implant, with less trauma to the patient as possible. The decision for surgery should be made to a comprehensive treatment plan, based on visual inspection, panoramic and periapical radiographs, in order to determine the anatomic limits, the occlusal conditions, the periodontal status and bone condition of the patient. This analysis can be enriched with lateral cephalometric radiographs and CT scans.

Precautions: required surgical techniques to achieve the DENTOFLEX Implants are highly specialized and involve complex procedures. Improper patient selection and / or techniques can cause the implant failure and loss of bone support.

Restrictions: DENTOFLEX the implant surgery is not indicated in cases where the bone is very small and does not provide height or width suitable for wrapping the implant.

Warnings: There may be lack of osseointegration or failure of the implant in cases where there is failure or poor bone quality, lack of available bone, poor oral hygiene, tobacco abuse or medical, uncontrollable conditions (such as blood disorder or diabetes).

The professional should handle the product only with titanium instruments or coated with titanium nitride. The surgery DENTOFLEX implant should be carried out by suitably qualified professional. Your box should only be opened in a sterile surgical field. Do not use the product if it is with the expiry date of the overdue sterilization or open packaging, violated or damaged. It is designed to USE ONLY ensuring peak performance and banned the reprocessing.

Storage: Implants DENTOFLEX, inviolate in their packaging, should be stored in a dry, cool and free from sunlight.

Contraindications: The DENTOFLEX implant should only be used as directed, so that any other uses are considered contraindicated or without scientific substrate that supports its use. Local or general infections can bring into play the purpose of the intervention; Inflammatory important local phenomena; Pregnancy; Immuno-depressive disorders; Bone immaturity; Profound mental illness.

Diseases of bone metabolism making random the support Mechanical expected with this type of implant.

Product disposal: In the event the product disposal need professional should follow the laws that apply to product disposal contaminated.

Side Effects: The risks of Implantologic procedure involving edema, bruising, bleeding and paresthesia (numbness) of the lower lip and chin region, the mandibular surgery, and surrounding tissues in the nose jaw surgery.

Although these symptoms are natural and temporary, in rare cases, the numbness may be permanent. They can also occur gingival-mucosal wounds, tissue reaction or infection. However, these diseases are usually treated successfully by means of appropriate medication.

Transportation: Triple unopened packaging of DENTOFLEX implants protects the product against drops and bumps. However, in long-distance transport, it is recommended that the package receive additional protection

Bone	Diameter of the implant	Mandible / Jaw										Counterside		
		Steel	2,6	2,8	3,2	3,4	3,7	4,0	4,2	4,3	5,0			
Type I & Type II	IH	3,50	*	*	*	*	*	*	*	*	*	*	*	
		3,75	*	*	*	*	*	*	*	*	*	*	*	
		4,00	*	*	*	*	*	*	*	*	*	*	*	
		4,25	*	*	*	*	*	*	*	*	*	*	*	
Type III & Type IV	HISW	4,50	*	*	*	*	*	*	*	*	*	*	*	
		5,00	*	*	*	*	*	*	*	*	*	*	*	
		6,00	*	*	*	*	*	*	*	*	*	*	*	
		3,75	*	*	*	*	*	*	*	*	*	*	*	
Type I & Type II	EH	3,50	*	*	*	*	*	*	*	*	*	*	*	
		4,00	*	*	*	*	*	*	*	*	*	*	*	
		4,50	*	*	*	*	*	*	*	*	*	*	*	
		5,00	*	*	*	*	*	*	*	*	*	*	*	
Type III & Type IV	TISW	3,25	*	*	*	*	*	*	*	*	*	*	*	
		3,75	*	*	*	*	*	*	*	*	*	*	*	
		4,00	*	*	*	*	*	*	*	*	*	*	*	
		4,50	*	*	*	*	*	*	*	*	*	*	*	
Type III & Type IV	xPCM	3,50	*	*	*	*	*	*	*	*	*	*	*	
		3,75	*	*	*	*	*	*	*	*	*	*	*	
Type IV		4,00	*	*	*	*	*	*	*	*	*	*	*	
		4,00	*	*	*	*	*	*	*	*	*	*	*	

o OPTIONAL

Location Tooth	Implants			
	IH	EH	Conical IH	Conical EH
11.21	4,0/4,25/4,5/5,0	3,75/4,0/4,5/5,0	4,25/5,0	4,3/5,0
12.22	3,5/3,75/4,0	3,25/3,75	3,50/3,75	
13.23	4,0/4,25/4,5/5,0	4,0/4,5/5,0	4,25/5,0	4,3/5,0
14.15.24.25	3,50/3,75/4,0/4,25	3,25/3,75/4,0	3,75/4,25	
16.17.26.27	4,25/4,5/5,0/6,0	4,5/5,0	4,25/5,0/6,0	4,3/5,0/6,0
31.32.41.42	3,5	3,25	3,5	
33.43	4,0/4,25/4,5/5,0	4,0/4,5/5,0	4,25/5,0	4,3/5,0
34.35.44.45	3,75/4,0/4,25	3,25/3,75/4,0	3,75/4,25	4,3
36.37.46.47	4,25/4,5/5,0/6,0	4,5/5,0	4,25/5,0/6,0	4,3/5,0/6,0

Location Tooth	Implants		
	Cone Morse	Short	Flex
11.21	3,5/3,75/4,25/5,0		4,25/5,0
12.22	3,5/3,75		3,75
13.23	3,75/4,25/5,0		4,25/5,0
14.15.24.25	3,5/3,75/4,25		3,75
16.17.26.27	3,75/4,25/5,0	5,0/6,0	4,25/5,0
31.32.41.42	3,5/3,75		
33.43	3,75/4,25		4,25/5,0
34.35.44.45	3,5/3,75/4,25		3,75/4,25
36.37.46.47	3,75/4,25/5,0	5,0/6,0	4,25/5,0

Bone	Diameter of the implant	Mandible / Jaw							
		Steel	2,6	Drill Conical 3,50	Drill Conical 3,75	Drill Conical 4,25	Drill Conical 5,00	Drill Conical 6,00*	
Type I & Type II	OC45WCM	3,50	*	*	*	*	*	*	
		3,75	*	*	*	*	*	*	
		4,25	*	*	*	*	*	*	
		5,00	*	*	*	*	*	*	
Type III & Type IV	OC15W30	3,50	*	*	*	*	*	*	
		3,75	*	*	*	*	*	*	
		4,25	*	*	*	*	*	*	
		5,00	*	*	*	*	*	*	
Type I & Type II	OC5WHI	3,50	*	*	*	*	*	*	
		3,75	*	*	*	*	*	*	
		4,25	*	*	*	*	*	*	
		5,00	*	*	*	*	*	*	
Type III & Type IV	OC5WHI FL	3,50	*	*	*	*	*	*	
		3,75	*	*	*	*	*	*	
		4,25	*	*	*	*	*	*	
		5,00	*	*	*	*	*	*	
Type I & Type II	CO45	3,50	*	*	*	*	*	*	
		4,30	*	*	*	*	*	*	
		5,00	*	*	*	*	*	*	
		6,00	*	*	*	*	*	*	
Type III & Type IV	CO5WHE	3,50	*	*	*	*	*	*	
		4,30	*	*	*	*	*	*	
		5,00	*	*	*	*	*	*	
		6,00	*	*	*	*	*	*	

* Drill not included in the Surgical Kit
o OPTIONAL

DENTOFLEX COM. E IND. DE MAT. ODONT. LTDA
Addr: Lino Coutinho, 634 - Ipiranga - 04207-000 - São Paulo - SP - Brazil
Phone: 55 11 2065 2155 - Fax: 55 11 2065 2153
vendas@dentoflex.com.br - www.dentoflex.com.br
Technician in Charge: Dr. Jorge Malatinho - CROSP 19628